



## 内窥镜手术模拟器开关机流程

### 一、开机流程

- 1、接通电源，打开机器背面的红色开关

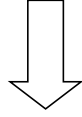


- 2、正确连接内镜，并打开前端的电脑开关，如下图

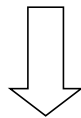

3、等待 1 分钟左右，将进入模拟训练系统登录界面，输入用户名：



admin，密码：admin，点击登录，进入系统

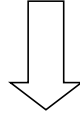

The screenshot shows a login page with the following elements:


- Please Login** (Title)
- 用户名 Login Name** (Text label) and **密码 Password** (Text label) with corresponding input fields.
- Can't access your account?** (Text link)
- 登录 Login** (Button)
- Select Font Size** (Section header) with radio buttons for **small**, **medium** (checked), and **large**.
- You are working offline on** (Text) with a cloud icon, **US061-02-18** (Text), and **关机 Shut Down** (Button).
- Change Settings** (Button)
- Visiting for the first time?** (Text) with links for **Register to use the system** and **Log in as a guest**.

4、点击资料库 (Library)，进入训练模式双击打开病例，开始操作

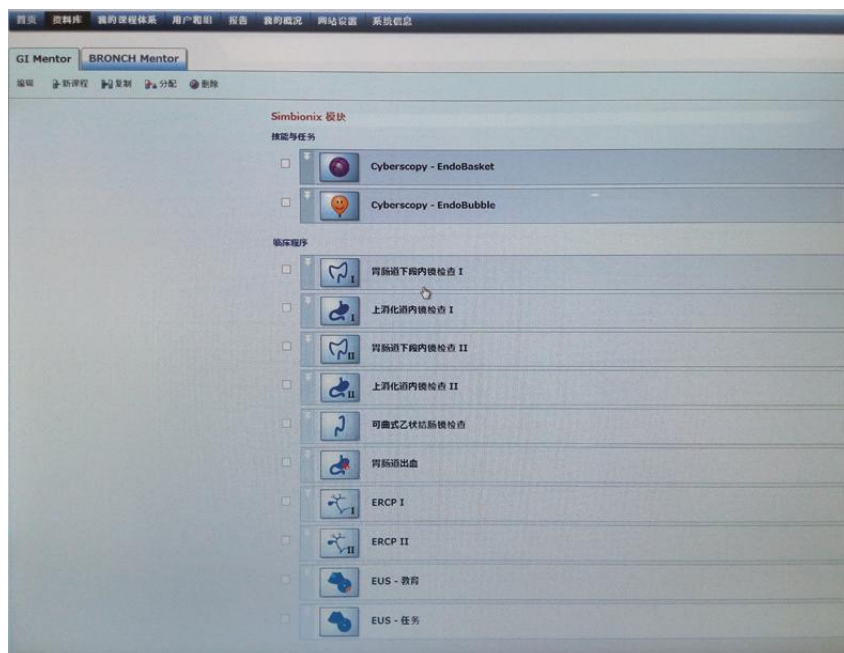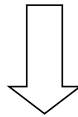


## 二. 关机流程

1、病例操作完成后，点击 finish 结束病例，返回资料库（Library）

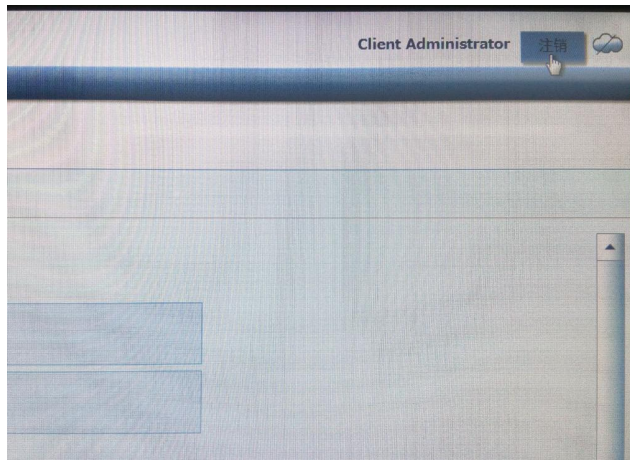

2、点击右上角“注销”（Logout），系统退回到登录界面

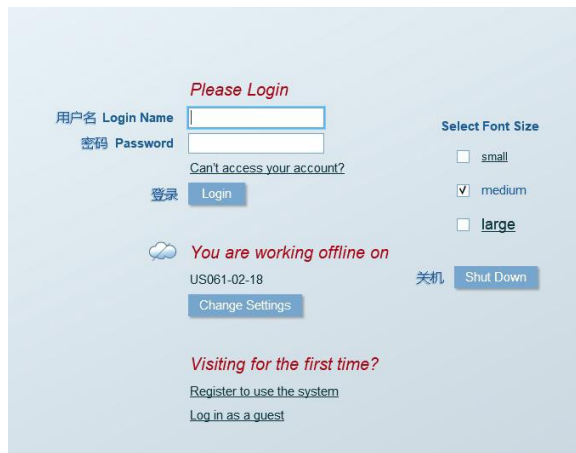

3、点击“shutdown”，系统弹出确认对话框，点击“OK”，确认关闭系统

4、关闭机器背面的红色开关

